

Epilepsy Diagnosis Aid

Guidelines for use

1 From Device Screen tap **EpilepsyDiagnosis** icon

2 To see the **Menu** either tap green bars at top left or swipe screen to right

3 To enter a new record tap **Add**

**This part of the app lets you enter information about patients to determine the probability of episodes being epileptic seizures.
If an answer is not known, tap NOT KNOWN and not NO**

4 Tap text box under **Patient Identifier** and keyboard will appear

Enter **Patient Identifier**

NOTE: you should not store any information which will enable a patient's identity to be discerned, either on your device or uploaded on to a server

Tap **Done** or ⇒

4 Enter whether **Eyewitness Available**, either in person or by phone

Tap ⇒

5 Enter **Gender**

Tap ⇒

6 Enter **Age** in years

Tap **Done**

NOTE: This app has not been tested on children less than 9 years. If age is less than 9 years you will be asked if you wish to proceed.

5 Ask about **Predisposing Factors**

Ask the 4 questions

A severe head injury is one with unconsciousness for 24 hours or more.

If ANY answers of Yes tap **YES**

If all answers of No tap **NO**

If not clear from the history tap **NOT KNOWN**

Tap ⇒

6 Now ask about **DURING** the attack. If no eye witness then there will be a lot of **Not Known**

7 Ask does any Color Change **DURING** the attack to Red or Blue or White or Grey.

If Red or Blue tap **Yes**

If any other response tap **No**

If uncertain tap **Not Known**

Tap ⇒

8 Ask if there is stiffening of the limbs. Demonstrate by stiffening your own arms.

If uncertain tap **Not Known**

Tap ⇒

9 Ask if there is shaking of the limbs. Demonstrate by jerking your own arms about once or twice per second

If uncertain tap **Not Known**

Tap ⇒

10 Ask if the tongue is ever bitten.

If uncertain tap **Not Known**

Tap ⇒

11 Ask if there is ever incontinence of urine

If uncertain tap **Not Known**

Tap ⇒

12 Ask if the head turns to one or other side. Demonstrate by turning your own head

If uncertain tap **Not Known**

Tap ⇒

13 Ask if eyes are **Closed**.

If open or rolling tap **No**

If uncertain tap **Not Known**

Tap ⇒

14 Ask if they can Communicate?

If uncertain tap **Not Known**

Tap ⇒

15 Ask whether **AFTER** the attack there is weakness of one or other side? If generalised weakness tap **No**

If uncertain tap **Not Known**

Tap ⇒

16 The app will then generate an **Epilepsy Probability Score**.

This screen provides guidelines for interpretation:

90 – 100 Almost certainly epileptic seizures

31 – 89 Uncertain

0 – 30 Almost certainly NOT epileptic seizures
(Uncertain if no eye witness)

If there is no eye witness and a score is < 30 then a yellow message will appear saying that the diagnosis is uncertain.

There is a box for Clinical Comments.

Tap the box to enter details, especially of clinical impression

17 Tap **View** to view and check entered record. If any mistakes tap **EDIT**

18 To Save record to device tap **SAVE**

19 To return to **Menu** either tap green bars at top left or swipe screen to right

20 Tap **View** to view records. A list of records on the device is displayed.

21 Tap an individual record to View it

22 Tap the ☒ at the top right hand corner to Delete this record from the phone. If it has been uploaded it will not delete it from the server.

23 To return to **Menu** either tap green bars at top left or swipe screen to right

24 Tap **Delete** button to see records prior to deletion. Delete individual records by tapping the ☒ beside the record. To delete all records, tap **Delete All** at the top right hand corner.

25 To return to **Menu** either tap green bars at top left or swipe screen to right.

26 To upload records to the server tap **Upload**. This will display a list of records on the phone with an indication whether they have been uploaded previously.

27 Tap the ↑ to upload individual records. Tap **Upload All** at the top right hand corner of the screen to upload all records.

NOTE: a wireless internet connection is required for this operation

28 To return to **Menu** either tap green bars at top left or swipe screen to right

29 **Settings** allows you to **Logout** of your account or **change password**

30 To return to **Menu** either tap green bars at top left or swipe screen to right

31 Tap **Exit** to leave the application

Vp/230414